





Hematology







The lifeline to better health

We are constantly developing innovative In-Vitro Diagnostic solutions to accurately uncover the truth hidden in a single drop of blood. Our actions help to understand the known and discover the unknown, leading to the earliest detection, treatment and monitoring of the health of patients all over the world. Our cutting-edge In-Vitro Diagnostic solutions provide the lifeline that connects doctors and clinicians to their patients to deliver a positive health outcome.

For Over 35 Years We Connect Healthcare Professionals with European Quality & Innovation in IVD

Cormay Diagnostics has a lot of experience and know-how in producing comprehensive range of market leading clinical chemistry and hematology reagents, together with innovations and patented technology have allowed us to deliver outstanding performance and reliability of our IVD solutions.

Our Cormay and Orphee IVD brands have become trusted by healthcare proffesionals from all over the world.

Flexible Fit Mythic Family analyzers suits in wherever needed

Perfect solution for wide range of hospital and clinical laboratories, satellite labs for specialized diagnostics, testing in clinics, hematology and oncology hospitals. Useful as a backup analyzer regardless the size of the laboratory. Thanks to small footprint, ideal solution for limited space.



We connect people in need with tests that can help them in their fight for health.

Our unwavering commitment to providing healthcare professionals with more effective tests that guarantee consistent, high-quality results and expert diagnosis ensures that patients around the world can count on a quick and healthy recovery.



The MYTHIC™ Family Multi-Patented Hematology Analyzers

Continuous Investment in Innovation, Technology, Research & Development.





TRUE 5+ DIFF TECHNOLOGY

Technology You Can Trust

When a diagnosis is accurate and made in a timely manner, a patient has the best opportunity for a positive health outcome because clinical decision making will be tailored to a correct understanding of the patient's health problem.



	Mythic 18	Mythic 3 CRP	Mythic 22 OT & 22 AL	Mythic 60 & 70
	IMPEDANCE 3 DIFF	IMPEDANCE 3 DIFF CRP + immunoturbidimetry	TRUE 5 DIFF Technology	TRUE 5+ DIFF Technology
GP/ Healthy				
Viral Infections				
Bacterial Infections				
Inflammatory diseases				
Allergies				
Parasites				
Oncology				
Hematology				



NATIVE CELLS MEASUREMENT FOR TRUE 5 DIFF SEPARATION

Blue-LED light source technology enhance scattering efficiency to reveal the finer cellular structure without damaging the cell. 445 nm wavelenght improve differentiation, identification and separation of five WBC populations.





CORMAY DIAGNOSTICS

Confidence In Your Clinical Decisions Trust MYTHIC[™] Multi-Patented Optical Technology

At Orphée, we understand the significance of technology in delivering fast and accurate diagnostic test results. Interferences are strongly dependent on the patient's health and can be very different from patient to patient. Mythics' innovative, patented single cell measurement technology ensures TRUE 5 DIFF and TRUE 5+ DIFF performance. Multipatented technology combined with our proprietary dynamic clustering algorithm delivers precise test results for confident diagnosis.

1 LED-BASED FLOW CYTOMETRY **SINGLE CELL TRUE 5 DIFF TECHNOLOGY**

Simultaneous measurements of light scattered in the Forward direction and Light Lost Axially of each cell crossing the interrogation zone generates the high definition matrix that enables high accuracy, direct measurement of each of the five WBC populations.





Patented hyrdofocus flow cell and improved injector technology employs a diluent sheath to drive and maintain sample flow across the cuvette through the detection area while reducing cellular damage.



3 DYNAMIC CLUSTERING

Mythic's 70 proprietry Dynamic Clustering Algorithm provides precise cellular classification specific to every patient's unique blood cells formation for accurate diagnosis. Dynamic clustering compensates for differences in WBC populations caused by variations in blood composition from patient to patient.



Incorrect thresholds lead to EOS incorrectly characterized as NEU resulting in a false "healthy" diagnosis.



Dynamic Clustering automatically identifies the correct thresholds for each patient and helps avoid incorrect diagnosis. Dynamic Clustering recognizes pathology samples that otherwise could be falsely flagged as a "healthy".

LYM	30 %	LYM	30 %
MON	5 %	MON	5 %
NEU	59 %	NEU	52 %
EOS	5 %	EOS	12 %
BASO	1 %	BASO	1%

WITH DYNAMIC CLUSTERING



Uncompromised, Comprehensive Results

Mythics' world-class hematology analyzers ensure TRUE 5 DIFF even TRUE 5+ DIFF TECHNOLOGY, multi-patented technology for more precise results and confidence in patients' diagnosis. Triple counting combined with authentic, not calculated, optical identification of WBC, provides real 5-part hemanalysis and in pathologies provides TRUE 5+ DIFF. Innovative technologies deliver comprehensive results and excellent performance at the same time.

Reliable 29 Parameters Results

Mythic 70 analyzer delivers reliable hematology results above standard 5-part differentation, with additional information from advanced research parameters: ALY %, ALY #, IMM %, IMM # to achieve two additional White Blood Cells populations in pathologies. Expanded platelets and red blood cells analysis guarantee comprehensive results for doctors and patients.

V	VBC:	RBC:		PLT:
WBC #		RBC		PLT
LYM %	LYM #	HGB		MPV
NEU %	NEU #	HCT		PDW*
MON %	MON #	MCV		PCT*
EOS %	EOS #	MCH		P-LCR*
BASO %	BASO #	MCHC	NEW	PLCC*
ALY %*	ALY #*	RDW-CV		
IMM %*	IMM #*	RDW-SD		

*Research Use Only Parameters



Uncompromised, Diagnostic Test Results You Can Trust

Orphée's innovative TRUE 5 DIFF technology combined with Dynamic Clustering intelligence ensure that when it comes to patient health, there is no compromise. Our Mythic's are designed to enhance your laboratory capabilities, producing high quality, accurate diagnostic test results in the most difficult of environments and on the most challenging patient samples.



Separation of WBC populations caused by temperature and time creates greater risk of incorrect cell type characterization. There is a greater potential for a "healthy" sample received for routine screening potentially being flagged as pathological.



	0 h	24 h 2-8 °C	24 h room temp.
LYM	18,3 %	20,9 %	20,6 %
MON	15,8 %	14,5 %	13,4 %
NEU	61,5 %	59,0 %	58,2 %
EOS	3,8 %	3,9 %	3,7 %
BASO	0,8 %	1,7 %	4,1 %

Mythic's 70 ability to keep cellular structural damage to a minimum makes it less sensitive to the changes to the blood sample caused by temperature or aging while dynamic clustering adapts to the decay of cells to maintain high quality performance.



	0 h	24 h 2-8 °C	24 h room temp.
LYM	21,8 %	20,8 %	22,1 %
MON	14,7 %	15,0 %	13,5 %
NEU	59,6 %	59,2 %	60,3 %
EOS	3,6 %	4,7 %	3,9 %
BASO	0,3 %	0,3 %	0,2 %











Mythic 3CRP Enhanced Diagnostics Performance

Mythic's 2 in 1 COMPLEX SOLUTION

Simulataneously performs blood & CRP measurements for Fast, Accurate & Comprehensive Whole Blood Screening.



Mythic 3CRP

Innovation That Plays A Leading Role In Reducing Antibiotic Resistance

Join 113 Countries and 40 Health Organisations by practicing antibiotic stewardship in support of the World Health Organisation's Call to Action on Antimicrobial Resistance (AMR) - 2021.



>99% Correlation to State of the Art Clinical Chemistry Analyzers CRP Diagnostic Test Results You Can Trust

Medical Practioners can now consider not prescribing antibiotics for patients who are unlikely to suffer from bacterial infection, while ensuring that patients who do require antibiotic treatment receive the appropriate antibiotics, at the correct dose and for the proper duration.





Your Cost Savings

Mythic's patented technology reduces reagents consumption and related waste by thousands of liters without any compromise in the accuracy of your diagnostic test result. Extremely low sample volume saves precious blood and expensive control material. Our Mythics provide affordable hematology testing for everyone.

Total reagents consumption up to 70% lower



Lyse consumption up to 95% lower



Extremely Economical Analyzers

Orphee's Mythic multi-patented technology not only delivers outstanding diagnostic performance but also has been proven to be very economical and reliable.

Improved Laboratory Efficiency

Patents reduce complexity of design and enhance longterm reliability. User friendly software does not create any problems for operators and makes hematology testing very fast, simple and easy

- Small sample volume
- One preventive maintenance per year
- **Reliable results**
- Compact analyzers
- Modern design
- **Reduced reagents** consumption





OnlyOne non toxic cyanide and azide free lyse helps to save the world for next generation.



Join to thousands of **satisfied customers**



Cormay Diagnostics Brands From Europe Answering your IVD Needs